With your inspiring rating, COSTWAY will be more consistent to offer you EASY SHOPPING EXPERIENCE, GOOD PRODUCTS and EFFICIENT SERVICE!

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
USER'S MANUAL

Shower Bench
BA1006

Dealer: This manual MUST be given to the user of the Shower Chair/ Bench.
 USER: Before using this Shower Chair/Bench, read this manual and save for future reference.

GENERAL WARNINGS

Rubber tips MUST be in contact with tub floor at all times.
 Always inspect the shower chair/bench to ensure that it is properly positioned and stable before using. DO not use if it is wobbly or unstable.
 Check rubber tips for rips, tears, crack or wear. If any of these conditions exist, replace rubber tips immediately.
 The Shower Chair/Bench has a weight limitation of 150 kg.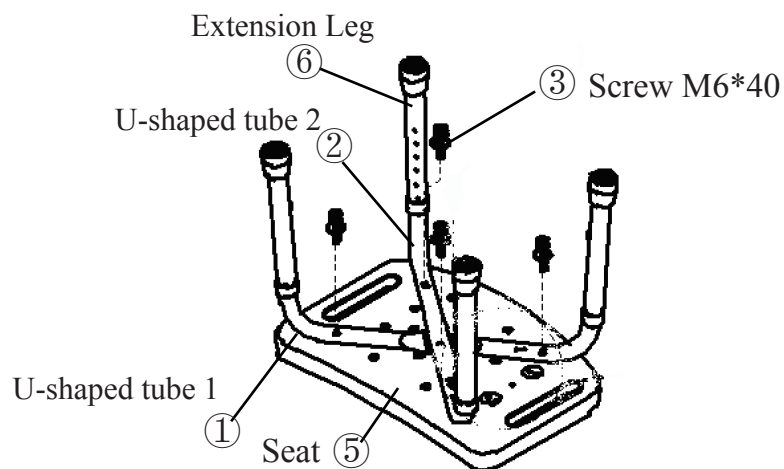

ASSEMBLING



1. Put seat upside down on the ground / floor.
2. Align U-shaped tubes on the seat and then attach 4 legs accordingly as shown in the figure.
 The U-shaped tube 1 is placed in the direction of seat plate position 1 and the U-shaped tube 2 is placed in the direction of seat plate position 2.
3. After aligning the holes in the overlapping area of the two U-shaped tubes, install screws and washers to fix the two U-shaped tubes on the seat plate, and then install the four legs.

Note: The incorrect placement of the 2 U-shaped tubes may cause unstable installation.

ADJUSTING LEG EXTENSION

1. Select a hole which is suitable to you on the lower leg frame.
2. Depress the snap button and slide the adjustable legs up or down to the desired height.
3. Ensure that the snap button is fully inserted into the tube's hole.
4. REPEAT STEP 2. 3. for the other legs

Make sure all four legs are adjusted to the same height so that the shower chair / bench sits level

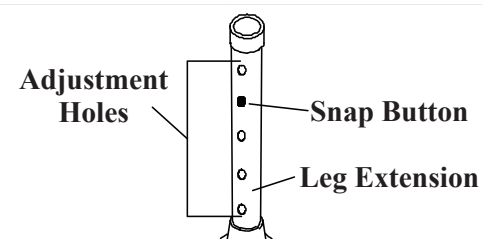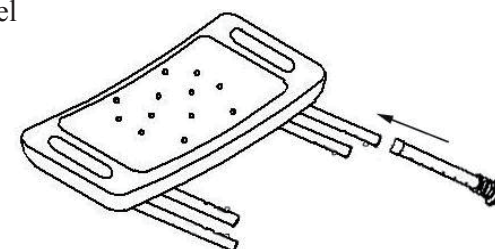Hardware Package

Item	Description	Unit
①	U-shaped tube 1	1
②	U-shaped tube 2	1
③	Screw M6*40	5
④	Washer	5
⑤	Seat	1
⑥	Extension Leg	4

NOTE: MAKE SURE ALL FOUR LEGS ARE ADJUSTED TO THE SAME HEIGHT AND THE SEAT IS OPENED TO FULL EXPANSION BEFORE USE.

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DE office: FDS GmbH, Neuer Höltingbaum 36, 22143 Hamburg, Deutschland