

COSTWAY®



USER'S MANUAL
Folding Rollator Walker
SP36907 BK/RE/NY

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION. PLEASE READ AND KEEP FOR FUTURE REFERENCE.

Please give us a chance to make it right and do better!

Contact our friendly customer service department for help first.

Replacements for missing or damaged parts will be shipped ASAP!

Follow Costway

Visit us: www.costway.com



Contact Us!

Do NOT return this item.

Contact our friendly customer service department for help first.



Before You Start

- ⊙ Please read all instructions carefully.
- ⊙ Retain instructions for future reference.
- ⊙ Separate and count all parts and hardware.
- ⊙ Read through each step carefully and follow the proper order.
- ⊙ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⊙ Always place the product on a flat, steady and stable surface.
- ⊙ Caution: Read the warnings carefully. Avoid kids get touch of small parts or plastic bag.

Contents

- 1. Introduction
- 2. Parts Description
- 3. Assembly instructions
- 4. Specifications
- 5. Safety tips
- 6. Warranty

1. Introduction

Aluminium rollator & transport wheelchair is suitable for everyday use. It can be used for both indoor and outdoor purposes. The lightweight design offers easy transportation and the sturdy aluminium frame gives stability and support when in use.

The Hybrid comes with high quality solid black puncture proof tyres. We have also included great features such as arthritic friendly loop lock cable brakes and ergonomic handgrips. The compact folding design helps with everyday storage and there are several height settings to ensure this product can be tailored for each individual.

Our products are always manufactured to the highest quality standards and are CE marked meeting all the necessary requirements under European legislation for medical devices.


2. Parts Description



3. Assembly Instructions

Please follow these simple steps to assemble -

1. Open up the main frame so the seat has locked into position as per the image below. Then place the main frame of the rollator on a solid level surface.

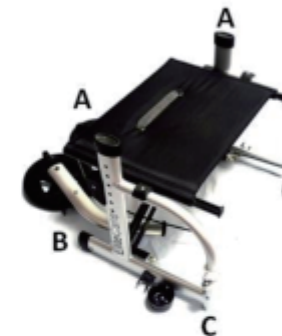

2. Insert the two height adjustable handle sections into the main frame marked "A" in the below image. Make sure the snap button has locked into position at your preferred height setting.



3. Insert the two rear brake wheels into the main frame marked "B" in the image above. Make sure the snap button has locked into position securely.



4. Insert the two front wheels into the main frame marked "C" and then turn the star knob shown in the image above clockwise until it locks into position. Make sure the screw is fully secured and not loose in anyway.



5. Finally just attach the footrests provided onto the front of the frame and the assembly is complete.

4. Specifications

Aluminium lightweight frame

Folding compact design

136kg Max user weight

Includes shopping bag

NW 8.5kg / GW 9.5kg

Dimensions (in CM) – Length = 100 / Width = 65

Available in a metallic silver paint finish

5. Safety Tips

It is important to keep yourself safe at all times when using your rollator especially when walking on slopes, inclines or while negotiating steps or kerbs.

Please select your route carefully to avoid uneven or soft ground wherever possible. Don't use the rollator on an incline if covered in snow, ice or a film of water or oil.

You must not make any unauthorised adaptations or changes to your rollator as this may cause it to become unstable. Practice on level ground as well as going up and down hills slowly.

Only one person can use the rollator in normal use. Never stand on the frame, seat or wheels when in use.

Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents. Users may put themselves at risk by improperly using their rollator.

The handbrakes should be used for slowing the rollator down when in use. Always remember to apply the parking brakes when stationary, except when storing the rollator for long periods of time as this could potentially damage or stretch the brake cables.

In the event of any damage or needed repair, immediately do not use your rollator for any purpose.

Always make sure you fully assemble and check your rollator carefully before use.

6. Warranty

There is a comprehensive twelve-month warranty from the date on which your rollator is delivered. The warranty covers the rollator for repairs or replacement during this period, this warranty is not transferable.

Should any parts of the product require repair or full replacement as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. The guarantee excludes tyres and all items that have been subjected to undue wear and misuse.

Under normal circumstances no responsibility will be accepted where the product has not been maintained in accordance with the manufacturer's recommendations.

The above warranty conditions only apply to brand new products.

In the event of your rollator requiring attention, please contact your service agent / dealer and give all relevant details so they can act accordingly.



Welcome to visit our website and purchase our quality products!

Your suggestions and comments for COSTWAY are really important to us!
We sincerely solicit you to go back to our shop and leave a good rating in just a simple click. It would be quite encouraging if you could kindly do so like below:

★★★★★ Great products so far

February 24, 2019

Great products so far. Fast delivery, easy setup, and working without any issues.

**With your inspiring rating, COSTWAY will be more consistent to offer you
EASY SHOPPING EXPERIENCE, GOOD PRODUCTS and EFFICIENT SERVICE!**

US office: Fontana, California

UK office: Ipswich



- * Reward Points
- * Exclusive Customer Service
- * Personalized Recommendations
- * Permanent Shopping Cart
- * Order History

5 Star Rating



We would like to extend our heartfelt thanks to
all of our customers for taking time to assemble
this product and giving us valuable feedbacks.

THANK
YOU
😊