



USER'S MANUAL

FOLDING WALKER

SP37508

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION. PLEASE READ AND KEEP FOR FUTURE REFERENCE.

Please give us a chance to make it right and do better!

Contact our friendly customer service department for help first.

Replacements for missing or damaged parts will be shipped ASAP!

Follow Costway

Visit us: www.costway.com



Contact Us!

Do NOT return this item.

Contact our friendly customer service department for help first.



Before You Start

- ⚠ Please read all instructions carefully.
- ⚠ Retain instructions for future reference.
- ⚠ Separate and count all parts and hardware.
- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.

Please read the manual carefully before assembling and keep it for future reference.


Product introduction: The support arm type walker frame tube is made of aluminum alloy, the black H frame and the grip tube are made of iron, the seat plate is PVC leather, the front wheel is 4 inches, the rear wheel is 5 inches with brakes, and it is equipped with hanging Bag. This product is suitable for the elderly, pregnant women, people with disabilities and other people with limited mobility to assist in walking. The frame seat plate can be folded.

Product Structure Diagram :



Preparation Before Assembly:

After unpacking the packaging box, first remove the protective bag and foam of the packaging. Please take care to protect all the parts in the box. If assembling on the ground, put paper or carpet on the ground first. To prevent scratches, pay attention to distinguish the parts and components, and then remove the inner carton to take out the parts and place them first, as shown in the figure below:

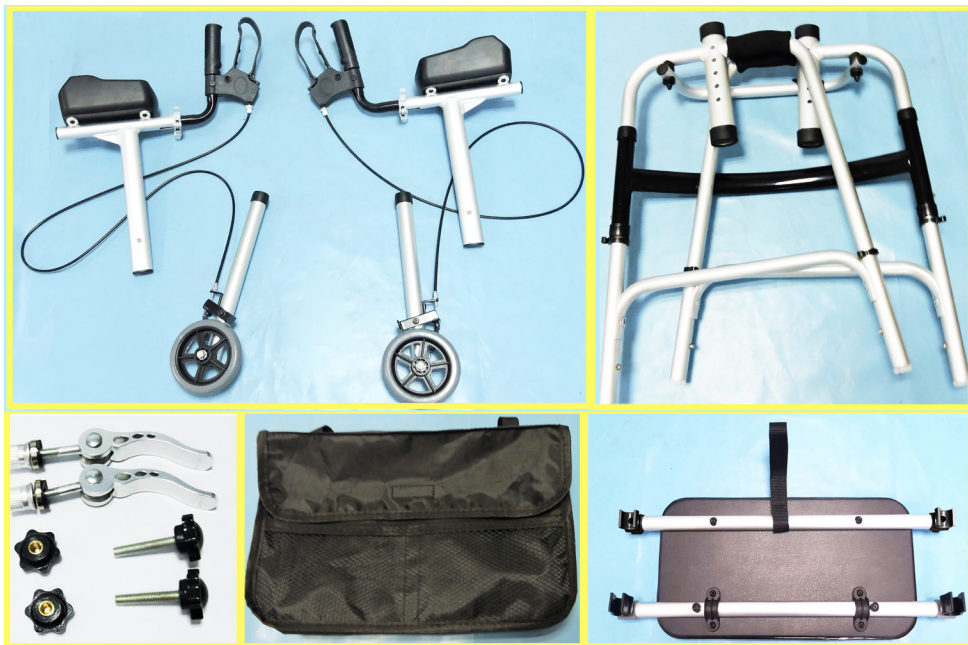

Assembly Instructions: please refer to the schematic diagram for assembly

Step 1: Open the frame, and the marbles of the small telescopic tube can be stuck in the marble hole of the large telescopic tube. (As shown in FIG. 1)

Step 2: Insert two front wheel components into the front tube positions of the frame until the marbles are stuck in the marble holes. (As shown in FIG. 2)

Step 3: Align the two rear wheel assemblies on the rear tube of the frame until the marbles are stuck. Pay attention to distinguish the left and right rear wheel components. (As shown in FIG. 3)



FIG. 1

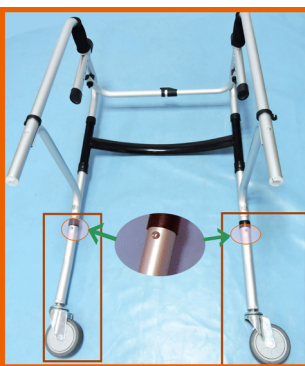


FIG. 2

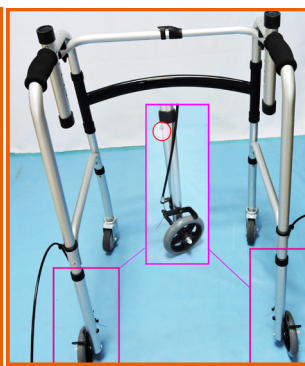


FIG. 3

Step 4: Insert the 2 armrest support components into the oval tube of the frame, and press and hold the marble to adjust the height to suit you. (Pay attention to distinguish the front and back direction)(As shown in FIG. 4)

Step 5: Two quick-release components are installed in the holes of the armrest support component and locked (please refer to the schematic diagram if the quick-release wrench is facing inward)(As shown in FIG. 5)

Step 6: Insert the two grip components into the armrest support component tube, and firmly press the quick release wrench to lock the grip components (if the lock is loose, adjust the quick release nut to tighten)(As shown in FIG. 6)



FIG. 4



FIG. 5



FIG. 6

Step 7: Fasten the brake cable with the plastic buckle on the frame. (As shown in FIG. 7)

Step 8: Clamp the plastic buckles at the front ends of the seat plate on the support tube of the frame, and then lock it with a Torx knob and Torx screws. (As shown in FIG.7 & FIG.8)

Step 9: Clamp the two plastic buckles behind the seat plate on the support tube of the frame. (As shown in FIG. 9)

Step 10: Install the hanging bag on the frame and stick it with Velcro. (As shown in FIG. 8)



FIG. 7



FIG. 8



FIG. 9

Check after installation:

1. Whether the arm-supported walker is installed correctly and whether the four legs are flat, please refer to the manual diagram for comparison.
2. Verify that all the accessories of the arm-type walking aid are fully assembled, and you can check for extra accessories.
3. Check whether there is any looseness in the installation of the arm-type walker. If there is any looseness, tighten the screws and nuts.
4. Check the appearance of all components of the arm-type walker for skew, and make appropriate adjustments.

Component Description:

The brake operation instructions are shown in the following figure:

A1: Normal use state without braking state

A2: Brake as needed during use

A3: Press down the brake handle to continue braking state

A3: Rotating nut can adjust the brake Tightness

A4: Rotating nut can adjust the brake Tightness.



A1



A2



A3



A3



A4

Product Care & Maintenance:

1. Do not scratch or scrape the surface of the product with hard objects to prevent material damage.
2. For daily cleaning and maintenance, apply neutral synthetic detergent, wipe with a damp cloth, and then wipe with a dry cloth.
3. Do not contact with strong acids and alkalis to prevent corrosion.

Storage Conditions and Methods:

After packaging, the arm-type walking aid should be stored in a clean room with a temperature not exceeding $-10^{\circ}\text{C} \sim 40^{\circ}\text{C}$, a relative humidity not exceeding 80%, no corrosive gas, and well ventilated. Keep away from moisture, light and heat during transportation.

Welcome to visit our website and purchase our quality products!

With your inspiring rating, COSTWAY will be more consistent to offer you
EASY SHOPPING EXPERIENCE, GOOD PRODUCTS and EFFICIENT SERVICE!

US office: Fontana, California

UK office: Ipswich



- * Reward Points
- * Exclusive Customer Service
- * Personalized Recommendations
- * Permanent Shopping Cart
- * Order History

5 Star Rating



We would like to extend our heartfelt thanks to
all of our customers for taking time to assemble
this product and giving us valuable feedbacks.

THANK
you
😊

