With your inspiring rating, COSTWAY will be more consistent to offer you EASY SHOPPING EXPERIENCE, GOOD PRODUCTS and EFFICIENT SERVICE!

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J. Cane Holder

DESIGN CONCEPT

Upright Rollator are designed to allow people to stand in a natural upright position thereby relieving the strain on his back and neck.

NEED TO KNOW

Don't Use The Device If You Have Any Of The Following Situations

- Visual impairments
- Severe balance issues
- Disabilities of Arm, Shoulder or Hand

Sit Down Must In Loop Locked Status

The brakes MUST be in the locked position BEFORE sitting on the seat. ALWAYS lock the brakes FIRST.

Timely Check

Make sure that all parts are secure and that moving parts are in good working order before each use. Periodically check nuts and bolts after use and re-tighten as appropriate.

This Is A Walking Aid, Don't Use As A Transportation Device.

Don't attempt to move the rollator while you or anyone is sitting on the seat.

WARNINGS

- >> Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- >> The rollator should be used on a level, dry ground. Ensure that you are accompanied by another person if possible. Practice using the rollator until you feel comfortable.
- >> When you use the rollator please keep your hands touching the brake. It can ensure you to stop the rollator immediately and effectively to reduce the risk of falling.
- >> Avoid sudden braking otherwise the rollator will slip especially on wet, smooth surface.
- >> If possible, please find somebody to help you to go up stairs, take the elevator.
- >> It is not recommended to use the rollator in BAD WEATHER (Rain/Storm/Lighting e.g.) or on the smooth/wet ground.
- >> Please slow down when start heading downhill.
- >> Don't push rollator while brakes are enabled. Brakes are used to keep the wheels from rolling; but it may not prevent the rollator from moving if it is pushed.
- >> Don't sit on the rollator while it is being loaded into a vehicle.

SPECIFICATIONS

Handle Height	40.5"-45.5"
Seat Dimensions	18" (L) x 10" (W)
Seat Height	21.5"
Weight Limit	300lbs
Overall Width	24.6"
Temperature	0—122°F
Wheels	10" + 8"

ASSEMBLY INSTRUCTIONS

Please follow the step by step instructions on the following pages to properly assemble your Upright Walker.

Insert the front wheels

Straighten the rollator

Press the seat until it's fixed

Insert the backrest and secure the knob by turning clockwise.

Insert the upright handle into the walker column, adjust the height you need until the button locked

Insert the stand handle

Insert the push handle into the steering column

Adjust the length and secure the knob by turning clockwise

Pull out red buttons and put the bag on

Insert the cane holder into the button

LOOP LOCK OPERATION

To Slow Down or Come to a Temporary Stop

Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.

To Lock Wheels in Position

Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.

To Release Wheel Locks

Pull up both loop locks with palms of hand to original position.