

**COSTWAY®**



**USER'S MANUAL**  
**Folding Walker**  
**SP0570**

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION. PLEASE READ AND KEEP FOR FUTURE REFERENCE.

**Please give us a chance to make it right and do better!**

Contact our friendly customer service department for help first.

Replacements for missing or damaged parts will be shipped ASAP!

**Follow Costway**

Visit us: [www.costway.com](http://www.costway.com)



# Contact Us!

## Do NOT return this item.

Contact our friendly customer service department for help first.



# Before You Start

- ⚠ Please read all instructions carefully.
- ⚠ Retain instructions for future reference.
- ⚠ Separate and count all parts and hardware.
- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.


## SAFETY PRECAUTIONS

Before use, follow the precautions below:

- 1.Ensure **all** the legs on the walker are set to equal heights and that this is the correct height for the user.
- 2.Ensure the brass push buttons have securely locked the legs in position.
- 3.Do not exceed the weight limit of 100kgs.
- 4.Inspect the walker regularly for signs of damage or wear(see care and maintenance section).

## USAGE

### 1. Setting the walker

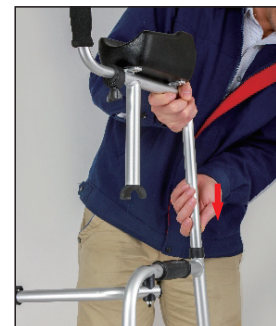

- 1.1 Take out the walker and all of the accessories from the carton.
- 1.2 Swing both side frames out away from the front cross brace until an audible "click" is heard.
- 1.3 Connect the left side front wheeled tube with the walker, press down snap button and slide on, line up pin and hole so pin pops through walker hole completely.  
Repeat for the right side front wheeled tube.
- 1.4 Position the left side forearm platform up, connect it with the walker, press down snap button and slide on, line up pin and hole so pin pops through walker hole completely. In the meantime, fit the clip on the frame tube.  
Repeat for the right side forearm platform.

## 2. Folding the walker



Find the plastic piece on the top of the front bar. Press down on the locking mechanism and fold the side frames in toward the front cross brace.

## 3. To adjust the handgrip



Unscrew the handgrip adjustable knob to release the handgrip. Move the handgrip horizontally to the required position. Screw the handgrip adjustment knob back in, ensuring the handgrip is secure and cannot move.

Repeat for the other handgrip. Ensure the handgrip is in the same position.

## 4. To adjust the height



- 4.1 Press the snap button, extend or retract the leg to raise or lower the height.
- 4.2 Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole. (An audible "click" will be heard)
- 4.3 Make sure that the walker is level.  
Repeat for the other legs, ensuring all legs are securely in position and all adjusted to the same height.

## 5. Fixing / Inactive adjustment



**Fixed Movement**



**Interactive Movement**

- 5.1 Press the folding button and fold the side frames slightly.
- 5.2 Fixed movement mode: press the snap button into upper hole (An audible "click" will be heard). It to be moved forward with lifting walker.
- 5.3 Interactive movement mode: press the snap button into lower hole (An audible "click" will be heard). It to be moved forward one side at a time, in step with the user.

## 6. To use the walker

The user(or carer) place their forearm on the platform and their hand around the handgrip. Repeat for the other arm and adjust the height if required and the position of the handgrip to suit the user.



## CARE AND MAINTENANCE

Maintenance should be done at least every six months.  
DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the walker.

- 6.1 Regular cleaning of the walker is recommended. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
- 6.2 Regular inspection of parts including hardware, brackets, and plastics, for deformation, corrosion, breakage, wear or compression is recommended. Replace the walker if any of these conditions exist.
- 6.3 If rubber tips are worn or cracked, replace the rubber tips IMMEDIATELY.

## Welcome to visit our website and purchase our quality products!

Your suggestions and comments for COSTWAY are really important to us!  
We sincerely solicit you to go back to our shop and leave a good rating in just a simple click. It would be quite encouraging if you could kindly do so like below:

★★★★★ Great products so far

February 24, 2019

Great products so far. Fast delivery, easy setup, and working without any issues.

**With your inspiring rating, COSTWAY will be more consistent to offer you EASY SHOPPING EXPERIENCE, GOOD PRODUCTS and EFFICIENT SERVICE!**

US office: Fontana, California

UK office: Ipswich



- \* Reward Points
- \* Exclusive Customer Service
- \* Personalized Recommendations
- \* Permanent Shopping Cart
- \* Order History

*5 Star Rating*



**We would like to extend our heartfelt thanks to all of our customers for taking time to assemble this product and giving us valuable feedbacks.**

THANK  
YOU  
😊