



COSTWAY®

Please give us a chance to make it right and do better !

Contact our friendly customer service department for help first.
Replacements for missing or damaged parts will be shipped ASAP !

US office: Fontana **UK** office: Ipswich **AU** office: Truganina
DE office: Hamburg **FR** office: Saint Vigor d'Ymonville **PL** office: Gdańsk



Contact Us!

Do NOT return this item.

Contact our friendly customer service department for help first.



US: cs.us@costway.com
UK: cs.uk@costway.com
AU: cs.au@costway.com

USER'S MANUAL

Toilet Safety Rails SP0579

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION. PLEASE READ AND KEEP FOR FUTURE REFERENCE.



Before You Start

- ⚠ Please read all instructions carefully.
- ⚠ Retain instructions for future reference.
- ⚠ Separate and count all parts and hardware.
- ⚠ Read through each step carefully and follow the proper order.
- ⚠ We recommend that, where possible, all items are assembled near to the area in which they will be placed in use, to avoid moving the product unnecessarily once assembled.
- ⚠ Always place the product on a flat, steady and stable surface.
- ⚠ Keep all small parts and packaging materials for this product away from babies and children as they potentially pose a serious choking hazard.

The design and manufacture are both conform to related product standards.

This product can be used for home health care and daily life. It does great favor in helping user standing up and sitting down. Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Warning

- Please ensure that you fully understand the warning as below before using the products.
- Please ask the customer services in case of any damage to the frame and rail.
- Check whether the fixing parts are correctly secured.
- If the rail is worn or invalid, please take it to a qualified technician for repair or replacement.
- Do not use on uneven, damp, or slippery ground.
- Clean regularly, wiping down with a soft cloth and a mild detergent/disinfectant solution.
- Do not use corrosive solution and abrasive cleaners.

Specifications

Max Load: 300 lbs (136kg)

Main Material: Carbon Steel


Handrail Adjustable Height: 24.8" - 30" (63cm - 76cm)

Handrail Width: 22.2" / 24.5" (56.5cm / 62cm)

Unpacking


- This product is supplied flat packed and requires assembly before use.
- No tools are required.
- Remove the product from its packaging and check for damage. If you see any damage or suspect a fault, do not assemble the product, contact the customer services immediately.

Check all parts are present- see content list as below:

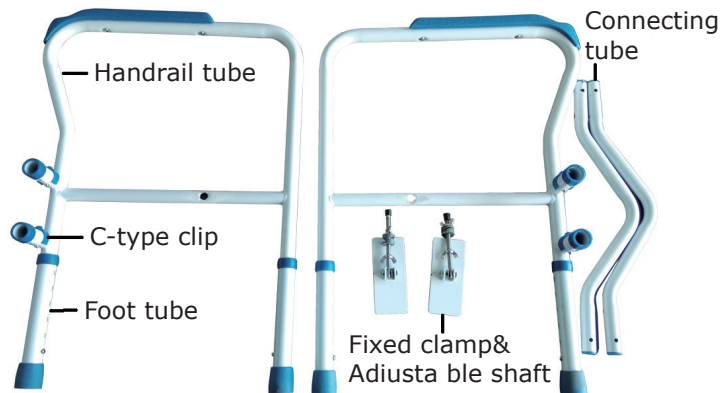

Assembly and Installation Instructions

Step 1: Detach 4 C-type clips from the tubes firstly.

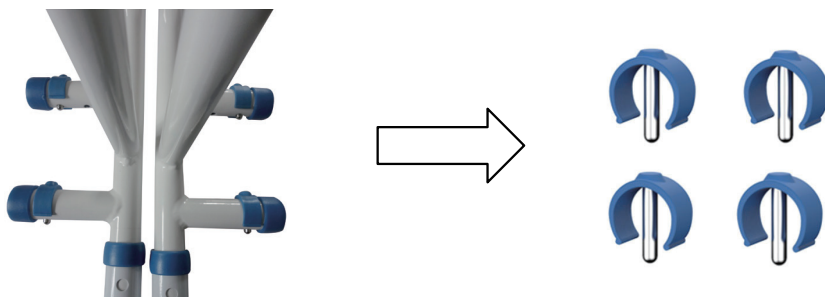

Step 2:
Insert the connecting tube into bushing and align the hole location.
Fix the C-type clips into 4 bushing hole sites.



Step 3: Detach the small plug, nut and arc sheet from the adjustment shaft.



Step 4: Insert the adjustment shaft into the hole of handrail tube.



Step 5: Assemble the nut and arc sheet as the below picture shows.



Step 6: Assemble the small plug at the end of shaft.



Step 7: Adjust the clamps to tighten the commode. See installation picture as below.

