



COSTWAY®



USER'S MANUAL
Toilet Wheelchair
SP36345

THIS INSTRUCTION BOOKLET CONTAINS **IMPORTANT** SAFETY INFORMATION. PLEASE READ AND KEEP FOR FUTURE REFERENCE.

Please give us a chance to make it right and do better!


Contact our friendly customer service department for help first.

Replacements for missing or damaged parts will be shipped ASAP!

Follow Costway

Visit us: www.costway.com



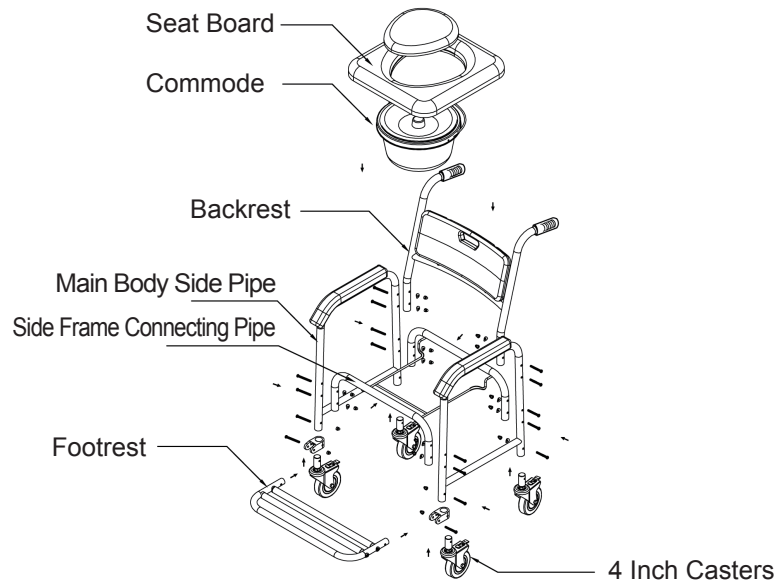

Installation Instructions

1. Install the commode bracket and the side frame connecting pipe together as shown in the figure, then tighten the main body side pipe and the connecting pipe with M6 screws, arc-shaped gaskets, and cap nuts in the direction of the arrow shown in the figure.
2. Connect the backrest frame to the main frame according to the arrow shown in the figure, and fasten them with M6 screws, arc-shaped gaskets and cap nuts.
3. Install casters and footrests in the direction of the arrows as shown in the figure, and fasten them with M6 screws. Then place the commode and seat board on the main frame, and insert the seat board into the main frame connecting pipe. The product is now ready to use.

Safety and Maintenance

1. Before use, check whether the seat board is locked. If you need to use the commode, you should remove the commode lid and lock the four wheels.
2. The user's weight should not exceed the rated load weight (100kg).
3. Please clean the toilet wheelchair after use. Avoid exposing the product to the direct sunlight and rain.
4. Frequently check whether all parts are loose or damaged to ensure safe use. The wheel rotation structure should be regularly lubricated.
5. Regular disinfection.